



Philips Respironics V60 Top Cover Replacement

This guide will show you how to replace the top cover of a Philips Respironics V60 ventilator.

Written By: Rie Kuraoka

M4 x 8 screw (x4)
(113 ± 11.3 N cm
 10 ± 1 in.-lbf)

Top cover



INTRODUCTION

If there has been damage done to the top cover of the Philips Respironics V60 ventilator, this guide will help you replace it.

The top cover just simply serves as a cover for the inside of the ventilator, making sure that the parts of the ventilator within are protected. Damage to this could also damage the inside of the ventilator so it would be good to replace/repair it as quickly as possible.



TOOLS:

- [Phillips #0 Screwdriver](#) (1)
-

Step 1 — shut down the ventilator



- Press and release the **ON/Shutdown** key, opening the **Shutdown** window in the screen.
- Select **Ventilator Shutdown**, which will shut down the ventilator.

⚠ Improper shutdown may cause a **Power has been restored** message the next time the ventilator is turned on.

Step 2 — disconnect from power




- Disconnect the ventilator from the AC power.

Step 3 — How to Disconnect a Philips Respironics V60



- Unplug the ventilator

Step 4

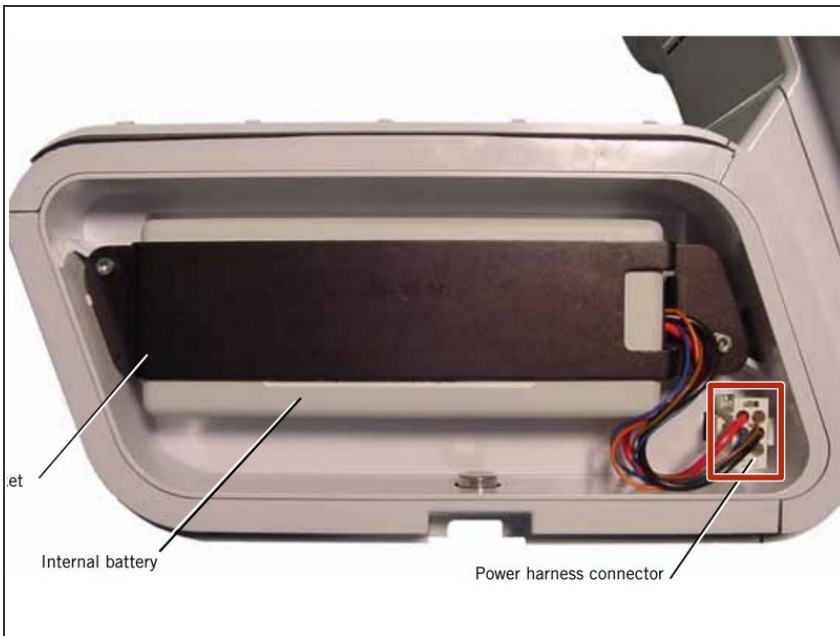


- Disconnect the internal battery from the power harness connector.

Step 5 — Top Cover



- Turn the captive D-ring fastener on the bottom of the panel counter-clockwise one-quarter turn and release.
- Then remove the right side panel.

Step 6

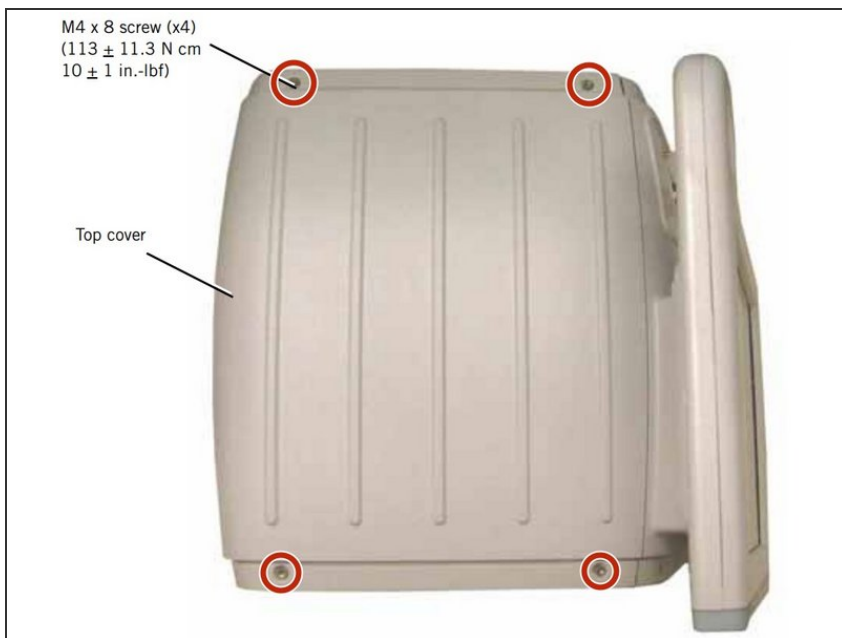

- Remove the 4 screws using a screwdriver.
- Remove the top cover.

To reassemble your device, follow these instructions in reverse order.